THUASNE®






SPORT SHOULDER SUPPORT 4

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de	Sport-Schulterstütze	7
nl	Schoudergordel sport	8
it	Tutore per spalla per uso sportivo	10
es	Hombreira para uso deportivo	11
pt	Ombreira para uso desportivo	13
da	Skulderbind sport	14
fi	Olkatuki urheilukäyttöön	16
sv	Axelskydd sport	17
el	Αθλητικός νάρθηκας ώμου	19
cs	Ramenní bandáž sport	21
pl	Orteza barku sportowa	22
lv	Sporta pleca polsteris	24
lt	Peties įtvarys sportui	25
et	Spordi õlatugi	27
sl	Opornica za ramo šport	28
sk	Športová ramenná bandáž	29
hu	Vállvédő sportoláshoz	31
bg	Раменна ортеза за спорт	32
ro	Orteză pentru umăr sport	34
hr	Sport remen za rame	36

SPORT

Instructions for use - Instructions d'utilisation - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Upute za uporabu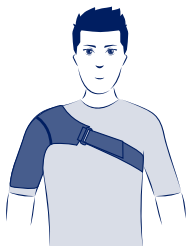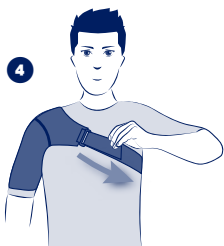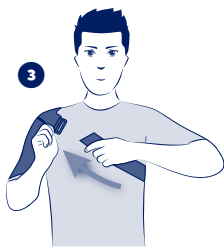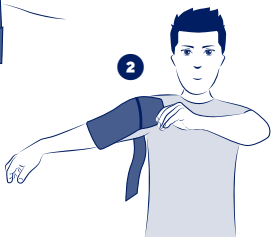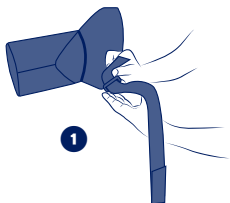

 cm
S 28 - 31
M 32 - 35
L 36 - 40

en	Biceps circumference
fr	Tour du biceps
de	Bizepsumfang
nl	Omtrek van de biceps
it	Circonferenza del bicipite
es	Circunferencia del bíceps
pt	Perímetro do bíceps
da	Overarmens omkreds
fi	Hauiksen ympäröymitta
sv	Mått runt biceps
el	Περιφέρεια δικεφάλου
cs	Obvod bicepsu
pl	Obwód bicepsa
lv	Bicepsu apkārtmērs
lt	Žasto apimtis
et	Ümber biitsepsi
sl	Obseg bicepsa
sk	Obvod bicepsov
hu	Bicepsz kerülete
bg	Обиколка на бицепса
ro	Circumferința bicepsului
hr	Opseg bicepsa

SPORT SHOULDER SUPPORT

Support level 1

Description/Destination

This device is a shoulder sling allowing the support of the shoulder. The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Polyester - polyamide - natural latex - synthetic rubber (SBR) - polyformaldehyde - elastane.

Properties/Mode of action

Elastic fabric covering the shoulder.

Support strap with an adjustable fastening.

Bilateral model.

Flat seams for increased comfort.

Anatomically-shaped to follow the movement of the shoulder.

Indications

Shoulder and/or elbow injuries or inflammatory conditions.

Shoulder subluxation or re-dislocation prevention (joint instability).

Shoulder dislocation.

Contraindications

Do not use the product if the diagnosis has not been confirmed.

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use in the event of complex or surgical shoulder fractures or rib cage fractures.

Do not use in certain post-operative conditions (multiple fractures, pin insertion, open fractures, etc.).

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

Consult a healthcare professional:

- In case of doubt before use,
- In case of uncertain diagnosis

Consult a healthcare professional and stop using the device:

- In case of discomfort, significant hindrance, pain, abnormal sensations,
- In case of changes in the device's performance.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Do not bring the product into direct contact with greasy substances (ointment, cream, etc.).

Prolonged immobilisation can sometimes cause muscle weakness.

If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

It is recommended to adequately tighten the device to achieve support without restricting blood circulation.

Do not wear the device when driving a vehicle.

Contains natural latex.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters, etc.) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Unfasten the self-fastening strap.

Pass the strap through the buckle on the back and attach it to the self-fastener ①.

Put the arm through the shoulder padding ②.

Pass the strap under the opposite armpit and pass it through the buckle on the front of the shoulder padding ③.

Adjust and attach the strap with the self-fastening closure ④.

Care/Maintenance

Product can be washed in accordance with the instructions shown on this leaflet and on the label. Close the self-fastening tabs before washing. Machine washable at 30 °C (delicate programme). Do not use detergents, fabric softeners or aggressive products (products containing chlorine). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2049201 (2023-10)
Thuasne SAS – SIREN/RCS Nanterre 542 091 186