


THUASNE®

en	Sport strapping ankle brace	4
fr	Chevillère strapping sport	5
de	Strapping Sport-Knöchelbandage	6
nl	Enkelbrace met elastische banden sport	8
it	Cavigliera strapping per uso sportivo	9
es	Tobillera strapping para uso deportivo	11
pt	Tornozeleira de "strapping" para uso desportivo	12
da	Ankelbandage sport	14
fi	Nauhallinen nilkkatuki urheilukäyttöön	15
sv	Ankelortos med elastiskt band sport	16
el	Αθλητική επιστραγαλίδα strapping	18
cs	Pásková kotníková bandáž sport	19
pl	Bandażowa orteza skokowo-stopowa sportowa	21
lv	Sporta potītes ortoze	22
lt	Apjuosiamas kulkšnies įtvaras sportui	23
et	Hüppeliigese sporditugiside	25
sl	Opornica iz trakov za gleženj šport	26
sk	Športová ortéza členka s páskami	27
hu	Pántos bokarögzítő sportoláshoz	29
bg	Ортеза за глезен с каишки за спорт	30
ro	Glezneră cu benzi de contenție sport	32
hr	Sportski steznik za gležanj s trakom	33

SPORT

Instructions for use - Instructions d'utilisation - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Istruções de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Upute za uporabu

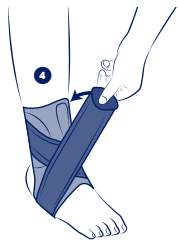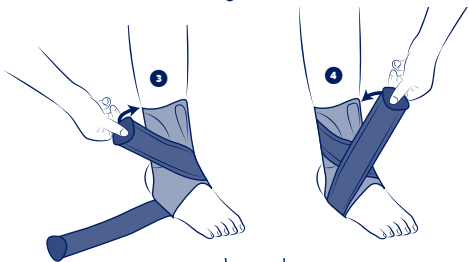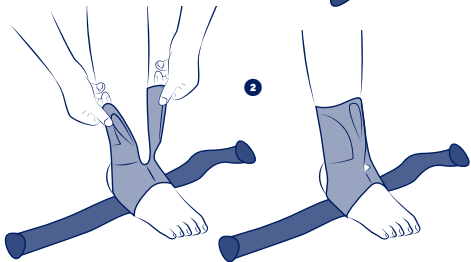cm
S 19 - 22
M 22 - 25
L 25 - 28
XL 28 - 30

en	Ankle circumference
fr	Tour de cheville
de	Knöchelumfang
nl	Omvang van de enkel
it	Giro caviglia
es	Circunferencia del tobillo
pt	Perímetro do tornozelo
da	Anklens omkreds
fi	Nilkanympärys
sv	Runt fotleden
el	Περίμετρος αστραγάλου
cs	Obvod kotníku
pl	Obwód kostki
lv	Potītes apkārtmērs
lt	Kulkšnies apimtis
et	Pahkluu ümbermõõt
sl	Obseg gležnja
sk	Obvod členka
hu	Boka kerülete
bg	Обиколка на глезена
ro	Circumferința gleznei
hr	Opseg gležnja

en

SPORT STRAPPING ANKLE BRACE

Support level 3

Description/Intended use

This device is a proprioceptive elastic ankle brace providing joint support.

The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table. This device can be used as a proprioceptive support for rehabilitation, particularly when resuming sporting activities.

Composition

Polyamide - synthetic rubber (SBR) - polyester - thermoplastic polyurethane - elastane.

Properties/Mode of action

Limitation of the heat loss by the synthetic rubber (Chloroprene).

Indications

Joint instability/laxity.

Mild ankle sprain (acute phase).

Contraindications

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use in the event of major venous thromboembolic history without thromboprophylaxis.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

Choose the appropriate size to fit the patient, referring to the size chart.

It is recommended to adequately tighten the device to achieve support without restricting blood circulation.

Consult a healthcare professional in case of uncertain diagnosis.

Consult a healthcare professional and stop using the device:

- In case of discomfort, significant hindrance, pain, abnormal sensations,
- In case of changes in the device's performance.

In case of doubt before any sports activity, consult a healthcare professional.

For hygiene, security and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

If the support is worn for prolonged periods of time, ensure that blood circulation is not impeded.

In case of history of venous or lymphatic disorders, consult a healthcare professional.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Possible risk of venous thrombosis.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Open the ankle brace and pull it on like a sock. ❶

Close the ankle brace using the clamping tab. ❷

Take the strap on the inside of the foot, pass it over the instep and secure it on the outside of the ankle. ③

Take the strap on the outside of the foot, pass it over the instep and secure it on the inside of the ankle. ④

Garment care

Product can be washed in accordance with the conditions shown on this instruction leaflet and on the label. Machine washable at 30 °C. Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2050801 (2023-12)
THUASNE UK Ltd
Thuasne SAS – SIREN/RCS Nanterre 542 091 186